

Operating Instructions:

❖ **Before Use:** Ensure the seatbelt is securely fastened and both armrests are folded down.

❖ **To Move:** Use the toggle switch located on the armrest or the remote control. Press in the direction you wish to travel.

❖ **At the Top:** When you reach the top, listen for a clicking sound.

Once the lift has fully stopped, gently press down on the handle connected to the seat to rotate it.

❖ **When Not in Use:** Always park the lift at its designated charging station.

Either at the top or bottom of the track. The display should show code **C1** or **C4**, indicating that it is charging.

❖ **Charging Warning:** If the display shows code **A1**, the lift is not charging and the battery may be dead when you next need it. To avoid this, make sure **C1** or **C4** is displayed whenever the lift is not in use.